

Rebokeh Magnifier App Notes

- **When you open the Rebokeh app, you'll see several circular icons surrounding the screen. To get them to disappear, tap the screen with one finger.**
- **To magnify, push one finger up on the screen. The longer you slide your finger the more it will magnify.**
- **To de-magnify, pull one finger down on the screen. The longer you pull your finger down the screen, the more it will de-magnify.**
- **The icons on the lower left side can invert the color, change the view to various color shades, change the aperture, and change the contrast.**
- **At the top right, one can enable their flashlight with a sliding scale.**
- **On the lower right side, you can reset the view or, with the bottom icon resembling a folder, you can add one pre-set mode that you'd like to save and use whenever you open the app for the future.**